

Instructions for IO KeepSafe™

IO KeepSafe™ can be used with all 3 EZ-IO® needles, as well as Illinois bone marrow needles.

1 Once you have successfully established your IO access, slide fastening clip out to the retention post.



1

2 Position IO KeepSafe™ over the top of the needle and tubing while ensuring the Velcro® faces the outside of the device. Once applied over the needle and tubing, gently hold pressure on the device.



2

3 Position the Velcro® strap around the patient's extremity and through the yellow loop to tighten.



3

4 Slide clip into place around the hub of the needle. The IO KeepSafe™ allows you to monitor the insertion site.



4



IO KeepSafe™

6550 Olan Drive
Canal Winchester, OH 43110

Visit us at www.IOKeepSafe.com
Email: IOKeepsafe@gmail.com

WARNING: Use with caution in the presence of edema

Single patient use only

Follow local protocols regarding use

Patent pending

IO KeepSafe™ is a trademark of IO KeepSafe
EZ-IO® is a registered trademark of Teleflex Incorporated
Velcro® is a registered trademark of Velcro USA